

Capo Natural

Minimally Invasive and
Scientifically Proven!



Capo Natural – Perfect Restorations

The right composite matters!

Capo Natural is an ultra-fine micro hybrid composite with improved abrasion characteristics. Due to the non-use of nanoparticles and the proven cell-friendliness, Capo Natural is absolutely biocompatible. Furthermore, the special color adjustments of Capo Natural allows you to work minimally invasively.

You reduce the risk of secondary caries without compromising the aesthetics and you can also use Capo Natural with other systems - no complete change needed.

But: You will go for a change, once you made the Capo Natural experience. Promised!

Indications

Direct anterior and premolar restorations of all cavity classes (Class I, II, III, IV and V according to Black).

- Minimally invasive veneers and fillings
- Indirect restorations such as inlays, onlays and veneers
- Extended fissure sealing on molars and premolars
- Endodontic posts
- Splinting of loosened teeth
- Shape and color corrections to improve the aesthetics

Benefits


- Extremely easy handling:
 - Easy shaping
 - Excellent polishability
 - Not sticky
- Biocompatible and gentle to the gums with excellent mechanical properties
- Highly abrasion resistant and antagonist friendly at the same time
- With the selectively adjustable color effect you achieve the optimal shade of the filling
- You create extraordinarily homogeneous restorations
- Works with most adhesive systems
- Minimally invasively
- Highly transparent enamel shades
- Economic



Vickers hardness	685 MPa
Compressive strength	452 MPa
Flexural strength	144 MPa
E-modulus	11,861 MPa

Physical properties


Scientifically tested by renowned institutions

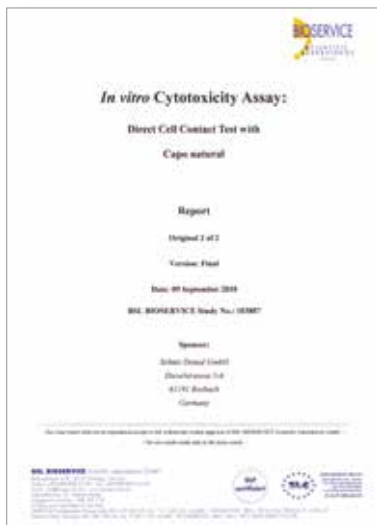

Cytotoxicity Test

Direct Cell Contact Test with Capo Natural. „In this study under the given conditions the test item did not show any reactivity (grade 0).“ The test showed that Capo Natural has no cytotoxic effects.


Source: BSL BIOSERVICE Scientific Laboratories GmbH, Planegg/Germany, Study-No: 103887

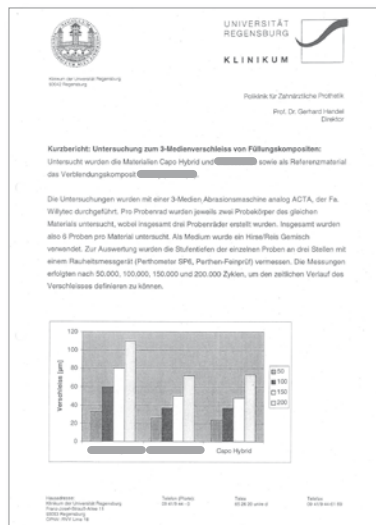

3 Media Abrasion Test




Material comparison of Capo Natural to other composites. Capo Natural showed the lowest abrasion in the process - even after 200.000 cycles.

Source: Klinikum der Universität Regensburg/Germany, Poliklinik für Zahnärztliche Prothetik

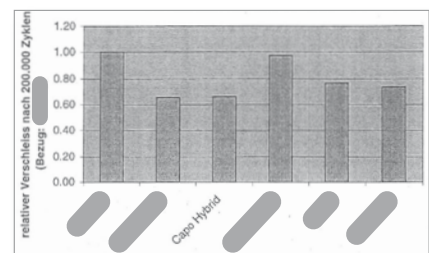

Results Abrasion Test

Capo Natural's high abrasion resistance is uniquely combined with its antagonist friendliness.

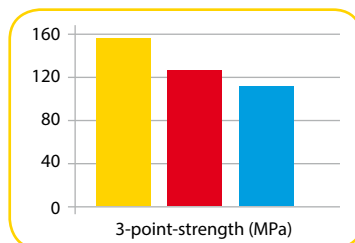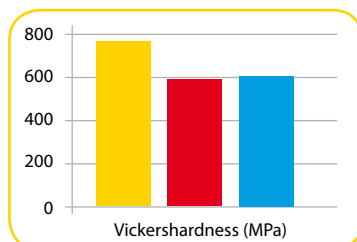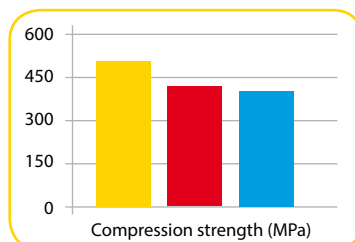

■ Capo Natural
■ Competitor A
■ Competitor B

Physical tests

Intensive tests by the independent Fraunhofer IKTS Dresden/Germany Institute compared three of the most significant indicators of physical properties of composites. Capo Natural had the best test results over all.

Source: Fraunhofer Institut für Keramische Technologien und Systeme, Dresden/Germany

For detailed information about the studies, please contact us

Phone: +49 6003 814-365

E-mail: export@schuetz-dental.de

We will provide you with the requested information!

Capo Natural

The system for perfect results

Capo Natural

A1	4 g syringe	Art. No. 232040
A2	4 g syringe	Art. No. 232041
A3	4 g syringe	Art. No. 232042
A3.5	4 g syringe	Art. No. 232043
A4	4 g syringe	Art. No. 232044
B2	4 g syringe	Art. No. 232045
B3	4 g syringe	Art. No. 232046
C2	4 g syringe	Art. No. 232047
C3	4 g syringe	Art. No. 232048

Special Colors

Incisial medium	4 g syringe	Art. No. 232049
Incisial clear	4 g syringe	Art. No. 232050
Incisial white	4 g syringe	Art. No. 232051
Bleach 1	4 g syringe	Art. No. 232052
Bleach 2	4 g syringe	Art. No. 232053



Capo Natural Set
6 x 4 g syringe,
Art. No. 232055

Capo Bulk Fill

Light-curing composite for posterior restorations. Suitable for layers with a thickness of up to 4 mm
2 g syringe, 10 application tips.



Art. No. 232060

Capo Universal

Has the same great features as Capo Natural. Available in one universal color for posterior restorations. High quality at a great price in a 6 g syringe.



Art. No. 232054

Capo Bond

Multifunctional, light curing 1-component primer and bonding system, ethanol-based. Extremely high bonding characteristics on dentin and enamel.
1 set consists of: 1 bottle Bond 5 ml, 1 dental brush holder, 50 disposable dental brushes, 2 mixing palettes



Art. No. 234014

Capo Etch

Safe and fast etching of dentin and enamel. High viscosity and excellent contrast to natural teeth, which makes the product easy and safe to apply.
1 set Etching Gel 2 x 2 ml, incl. 20 cannulas



Art. No. 234013

Capo Bond Activator

Makes Capo Bond a self cured material. This product offers a high degree of safety - even with deep cavities.
5 ml



Art. No. 234402

Capo Flow

Flowable, light-curing composite. Very good adaption, even with undercuts. 1 set consisting of: 6 x 1 g syringes in the colors A2/A3/A3.5/A4/B3/C3, 30 cannulas



Art. No. 234012

