AlphaDie MF

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Directions for use

Range of Applications

AlphaDie MF is excellent for fabricating precision models for all types of crown and bridgework as well as for dimensionally stable reproduction of shoulders and partial crown preparations.

AlphaDie Separator must be applied when casting polyether impression, resp., duplicating materials.

Do not use with alginates and hydrocolloid impression materials.

Properties

AlphaDie MF is flowable, accurate in detail and dimensionally stable. It has high edge strength and abrasion resistance and is easy to trim and saw.

Comprises

AlphaDie MF is supplied in glass bottles (200 ml base material, 100 ml hardener), together with 400 g filler. A dispensing scoop for the filler, mixing/dispensing vessel and spatula are included in the set.

Mixing AlphaDie MF

Briefly shake the bottles containing both the base material and hardener prior to mixing. Note the coloured marking on the cap. Close the glass bottles carefully immediately after use. **Do not allow AlphaDie MF to come into contact with water** (foam).

1. Fill the required amount of base material in the dispensing and mixing container supplied. Then add the correct quantity of filler and thoroughly spatulate the mixture.

Add the correct quantity of hardener and spatulate the mixture again thoroughly.

Mixing ratio:

Base: hardener = 2 : 1.

Approx. 15 g AlphaDie MF and filler is required for a full dental arch.

10 g base

1 level measuring scoop of filler (approx.

14 g)

5 g hardener

2. After mixing for approx. 30 sec., pour the AlphaDie MF in a thin stream into the cleaned and dried impression. The material remains flowable for approx. 2 min. at 20°C.

3. In order to prevent incorporation of air voids, AlphaDie MF may be hardened for 15 mins., after pouring, in a dry pressure vessel at 2-4 bars.

AlphaDie MF is sufficiently hard after 1 hour that the arch can be trimmed, drilled and ground.

CAUTION!

Do not overheat AlphaDie MF. Steam-clean AlphaDie MF (max. 15secs.).

Disposal

Dispose of in a special refuse tip, in accordance with local regulatory requirements.

Carefully carried out working procedures ensure that the die meets the most stringent requirements.

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