

MICROCLEAN table

The table assumes that 150 g of pins and 150 ml of water are used.

cleaning fluid - orange (universal), 500 ml, order no.: 678855, for following applications:

- Removing oxides from chrome cobalt, titanium and non-precious metals after sandblasting them to remove the investment material.
- Removing oxides from titanium and non-precious metals after firing porcelain.
- Removing cement from bridgework.
- Removing plaster from acrylic.

Cleaning in the large vessel, order no.: 678820

The amounts are always 150 g pins and 150 ml water

Time Minutes	Pins Description	Amount Fluid	Object/Application	Outcome
45 - 60	short/long	70 ml	Titanium,CrCo frameworks	Matte, silvery sheen
20 - 40	short	70 ml	Removing plaster from acrylic	Removes up to 98%
30 - 45	short/long	70 ml	Removing cement from crowns	Cement is released. Soft material must be scratched out or rinsed out with water if necessary.

Cleaning in the small vessel, order no.: 678864

The amounts are always 15 g pins and 15 ml water

Time Minutes	Pins description	Amout	Object/Application	Outcome
5 - 10	short	5 - 10 ml	Titanium,CrCo frameworks	Matte, silvery sheen

cleaning fluid - green (acidic), 500 ml, order no.: 678857 for the following applications:

- Removing oxides from precious metals after deinvesting.
- Removing oxides from previous metals immediately after firing porcelain.
- Cleaning deposits and calculus off dentures.
- **Do not keep needles in the liquid for longer than one day!**

Cleaning in the large vessel, order no.: 678820

The amounts are always 150 g pins and 150 ml water

Time Minutes	Pins Description	Amount Fluid	Object/Application	Outcome
30 - 45	long	40 ml	Cleaning deposits and calculus off used restorations	Cleans the restoration by up to 98% to regain its sheen
5 - 10	short/long	40 ml	precious metal framework	Matte sheen
40 - 50	short/long	40 ml	non-precious metal framework	98% sheen

Cleaning in the small vessel, order no.: 678864

The amounts are always 15 g pins and 15 ml water

Time Minutes	Pins Description	Amount	Object/Application	Outcome
5 - 15	short	5 – 10 ml	precious metal frameworks	Matte sheen

* Long pins = polishing pins, long, 150 g Order No.: 678865
Short pins = polishing pins, short, 150 g Order No.: 678866

Long pins = polishing pins, long, 200 g Order No.: 678856
Short pins = polishing pins, short, 200 g Order No.: 678861