



**The most important facts in short:**

**Take the time to read the  
Microclean instruction  
manual**

**Never use more than 150g polishing pins and 150ml water.**

**Each restoration must be cleaned separately .**

**The lid of the cleaning vessel must remain closed during the complete cleaning procedure (while the system is running).**

**In order to avoid rotation of the cleaning vessel, set a paper towel between unit and black beaker mount.**

**When exchanging the cleaning fluid, set the magnet against the outside of the vessel bottom, thus holding the pins. Clean the pins with water.**

**If the pins are dirty or tarnish, clean them for 30 min. in cleaning fluid or exchange them completely.**

**Please note the system table (see reverse side) and follow it !!!**

**Avoid direct contact of skin and eyes with the fluid, as it may irritate them.**

**cleaning fluid (acidic):**

- **cleaning of dentures, removing deposits and tartar**
- **removing oxides from precious metals**

**cleaning fluid (universal):**

- **removing plaster from acrylics**
- **removing oxides from non-precious metals and titanium**