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### Metall-Kunststoff-Verbundsystem

#### Sebond Smart

#### Anwendung

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#### Sebond Smart

#### Application

Sebond smart is used as a bonder between acrylics / composites and all types of metal.

After finishing, sand-blast the surface to be veneered with aluminium oxide (50 µm, 2 bar). We recommend to blow off the surface activated by sand-blasting with oil-free compressed air. Alternatively, the sand-blasted surface can be cleaned with steam or acetone. If one of these two techniques is used, let the metal surface dry well before proceeding. Next, apply a generous amount of Sebond Smart with a brush and let it sit for ca. 1 min. If necessary, apply a second layer.

#### Storage

Sebond should be stored at 10 - 25°C. Close the bottle properly immediately after use and protect against the effects of direct light.

#### Shelf life

The maximum shelf life is printed on the label of each bottle. Do not use after the expiry date.

Each bottle contains: 5 ml

#### Guarantee

Our technical instructions, regardless of whether they are provided verbally, in writing or during practical demonstrations, are based on our own experience and should only be considered guidelines. As our products are subject to continued development, we reserve the right to modify them. dialog complies with EN ISO 10477.

#### Note

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

#### Side-effects

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

#### Contra-indications / interactions

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.

#### Troubleshooting List:

Trouble	Cause	Remedy
Bond does not last	Material coat uneven and patchy	Make sure to apply the material in an even, uniform coat. If necessary, apply a second layer.
Material evaporated or dried up	Bottle was not closed correctly	Always close bottle correctly after each use.

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