dialog Pastenopaker

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Instructions for use

The framework is prepared with the metal-resin bonding system Sebond Smart.

dialog paste opaque is applied with a stiff, short-haired brush. Incomplete opaque coverage negatively affects the shade. At least two coats are required to ensure complete coverage. The flow of the opaque is improved by mixing it on a mixing pad. The paste should be applied especially thinly to retention areas. Each layer is polymerized separately.

Spektra LED Incisal / Dentin: 30 sec.
 Cervical: 1 min.
Spektra fast: 3 min.
Spektra 2000: 3 min.
PLC Spektra: 3 min.

If the opaquer layers are applied too thickly, it prevents optimum polymerization and weakens the bond. Extending the polymerization time does not increase the depth of polymerization.

Useful tips
The surfaces of the framework should be cleaned of any grease and polish residue. The opaquer should only be applied in very thin layers.

Working time for the materials
Depending on the lighting conditions 1-3 min.

Storage
The material should be stored at 10-25°C. Close the syringe properly immediately after use and protect against the effects of direct light.

Shelf life
The maximum shelf life is printed on the label of each syringe. Do not use after the expiry date.

*) Vita® is a registered trademark of Vita® Zahnfabrik H. Rauter GmbH & Co. KG, Bad Säckingen, Germany.

Vita®** shade	Paste opaque		Incisal	Composite		
	Cervical	Dentina		Cervical	Dentina	Incisal
A 1	P-H 1	P-D 1	P-S 1	HA1	DA1	S58
A2		P-D 2		HA2/3	DA2	S58
A3	P-H 2	P-D 3		HA2/3	DA3	S59
A3,5		P-D 4		HA3,5	DA3,5	S59
A4		P-D 4		HA4	DA4	S60
B1	P-H 1	P-D 5		HB1	DB1	S57
B2		P-D 6		HB2	DB2	S59
B3	P-H 2	P-D 7		HB3	DB3	S59
B4		P-D 7	HB4	DB4	S59	
C1	P-H 3	P-D 8	HC1	DC1	S60	
C2		P-D 9	HC2	DC2	S59	
C3		P-D 9	HC3	DC3	S59	
C4	P-H 4	P-D 8	HC4	DC4	S60	
D2		P-D 10	HD2/3	DD2	S60	
D3		P-D 2	HD2/3	DD3	S59	
D4		P-D 4	HD4	DD4	S59	

*) Vita® is a registered trademark of Vita® Zahnfabrik H. Rauter GmbH & Co. KG, Bad Säckingen, Germany.

Light-curing unit	Paste opaque	dialog Vario / dialog Vario Flow		dialog Occlusal	
		Intermediate polymerization	Final polymerization	Intermediate polymerization	Final polymerization
Spektra LED	1 min.	30 sec.	3 min.	30 sec.	3 min.
Spektra 2000 fast	3 min.	90 sec.	11 min.	90 sec.	11 min.
Spektra 2000	3 min.	90 sec.	16 min.	90 sec.	16 min.
DC-XS	90 sec.	30 sec.	540 sec.	30 sec.	480 sec.
Uni-XS / HiLite	90 sec.	90 sec.	360 sec.	90 sec.	360 sec.
Spektramat	2 min.	1 min.	10 min.	1 min.	10 min.
GC Labolight LV-II / III	1 min.	0,5 min.	18 min.	0,5 min.	18 min.
Solidilite EX / V	1 min.	90 sec.	16 min.	90 sec.	16 min.

CE 0297

MANI® SCHÜTZ DENTAL

Schütz Dental GmbH · Dieselstr. 5-6 · 61191 Rosbach/Germany
Telefon: +49 (0) 6003 814-0 · Telefax: +49 (0) 6003 814-906
www.schutz-dental.de · info@schutz-dental.de