## ES

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## E

UNIDESDA introduces **IDOFAST**, self cure acrylic resin for denture repair and partial and total relining of removable dentures. IDOFAST acrylic resin is from the IDO range, using a formula to achieve the best results in an easy and practical way. It is available in pink, pink veined and clear, offering hardness and colour stability unaffected by time due to its amine tertiary free composition.

Idofast acrylic resin is cadmium free.

### Instructions:

Free dosing of IDOFAST is possible.

Recommended mixing ratio: 7 gr. of powder - 5 ml of cold cure liquid.

- Mix thoroughly with a spatula at room temperature (21-23°C) and cover it. (Pour the liquid in a mixing jar, then add the powder. Stir sufficiently (30 seconds) to assure wetting of all powder particles, mix thoroughly for 30" with a spatula after this, cover mixing jar and allow material to reach pouring consistency (approx. 2 minutes at room temperature of 23°C).

Its special formulation creates a fast fluid mass suitable both pouring technique and conventional processing dough technique.

- Pouring time at 23°C: Handling time is subject to surrounding room temperatures; at 23°C IDOFAST is pourable for 2-3 minutes after mixing. (During the flow and the pour time, the material can be casted in a key or brushed).

- Modeling time at 23°C: 3 - 5 minutes (during the modeling time, the material can be molded in a key or modeled)

- Dough time at 23°C: 6 - 7 minutes.

- Resin self-cures in 15 min. at 50°C with 2 bars pressure in a pressure apparatus hydroflask.

- After mixing, resin self-cures in 30 min. at 23°C if a pressure unit is not available but using thicker layers of IDOFAST.

**Note:** Do not introduce the liquids into the fridge after or before use it. Store it always at room temperature.

### PRECAUTIONS:

This product is not resistant to chemical oxidant agents or strong acids.

During processing of cured material avoid breathing in dust and fumes.

The product is not visible by x-rays.

Do not reuse this product.

Product specifically designed for removable prosthesis

Do not modify mixing ratios or curing times or temperatures as this may affect the final result. Do not mix the product (powder / liquid) with other different products.

Do not contaminate during processing.

Acrylic resin: In case of an allergic reaction, stops further application and seek medical advice

### HAZARD DATA SHEET

**Powder:** Store in a cool, dry place away from direct sunlight.

R 42/43: May cause sensitization by inhalation or skin contact

R 36/37: Irritant for eyes and respiratory vies

**Liquid:** Store below 30°C in a dry place, away from direct sunlight.

R11: Highly flammable.

R36/37/38: Irritant to skin, eyes and nose.

R43: May cause sensitization by skin contact

S9: Store in a well-ventilated place.

S16: Keep away from sources of ignition – No smoking

S26: In case of contact with eyes, wash immediately with water and seek medical assistance.

S29: Do not empty into drains

### INFORMATION FOR THE PATIENT

The patient should be informed of the following points:

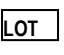
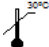
- Normal oral hygiene must be maintained.
- The material is not visible at x-ray.
- Material is resistant to abrasion but heavy bites should be avoided.
- In case of an allergic reaction, seek medical advice.

The material is for dental use only.

The manufacturer is not responsible for damage caused through improper use or any other use than those explains in these instructions.

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### Explanation of symbols used on the label

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