

ReVeneer

Rescue for
ceramic fractures





Initial appearance



Final appearance

The solution for ceramic fractures!

Two unique product advantages lead securely to an aesthetically impeccable result.

ReVeneer is an ingenious system consisting of primer, base paste and composite. In most cases, it saves you from having to refabricate a crown or bridge.

The ReVeneer system provides an easy solution with instant relief for dentist and patient: aesthetic, lasting, safe and acid-free.

Advantages of ReVeneer

- No acid etching (neither hydrofluoric nor phosphoric acid)
- Safe for intraoral use
- No silane as primer (hydrolytic degradation of this primer in the mouth is possible)
- Extremely high bonding strength
- Time saving
- Can be used universally
- Tested and optimised on all currently used ceramics (feldspath and glass ceramic, press ceramic, zirconia, aluminium oxide)
- Use of conventional curing lamps (halogen, LED, plasma light) possible

Gerrit Meyer MDD of the University Clinic Mainz/Germany is enthusiastic:

„The green color of the base paste proved to be very positive as it provides a nice contrast to the ceramic and thus makes it easier to work precisely“



Because of ReVeneer Base's green color, precise application is possible. The change in color from green to clear indicates when curing is complete.

Ceramic repair in practice with ReVeneer

Clinical case of a ceramic fracture

- 1 Initial appearance**
- 2 Roughening**

Roughening the surface with fine diamonds
- 3 Use the ReVeneer primer**

Drying time two minutes
- 4 Applying the ReVeneer opaquer**

Applying and curing the first thin layer
- 5 Applying the ReVeneer opaquer**

Applying the second layer
- 6 Curing time 60 - 90 seconds**
- 7 Applying ReVeneer base paste**
- 8 Brush on ReVeneer base paste**
- 9 Curing time 60 - 90 seconds**

After curing: Application of composite according to the instructions
- 10 Final appearance**

After polishing the composite restoration

Note: This is not an instruction leaflet. Please adhere to the instructions provided with the material. The responsibility for the treatment remains with the attending dentist.

Photos: MDD Gerrit Meyer, University Clinic Mainz/Germany; Guido Schwalm, Dentist, Rosbach/Germany

Packaging units ReVeneer system

Sets

Art.No.	Description	Content
232205	ReVeneer Adhesive Set	1 syringe (1 g) ReVeneer base 1 bottle (3 ml) ReVeneer primer 12 application tips (0.9 x 13 mm) 1 brush holder 6 disposable brushes
232200	ReVeneer Repair Set	1 syringe (1 g) ReVeneer base 1 bottle (3 ml) ReVeneer primer 12 application tips (0.9 x 13 mm) 1 dental brush holder 6 disposable brushes 1 syringe (3 g) opaquer universal light 1 syringe (3 g) opaquer universal dark 6 Fillis composite A3.5 with 0.3 g 3 Fillis composite A3 with 0.3 g 2 Fillis composite A2 with 0.3 g 1 Filli composite B2 with 0.3 g

Refills

Art.No.	Description	Content
232210	ReVeneer base	1.0 g
232211	ReVeneer primer	3.0 ml
232220	ReVeneer opaquer universal light	3.0 g
232221	ReVeneer opaquer universal dark	3.0 g
232231	ReVeneer composite A2	3 Fillis with 0.3 g
232232	ReVeneer composite A3	3 Fillis with 0.3 g
232233	ReVeneer composite A3.5	3 Fillis with 0.3 g
232236	ReVeneer composite B2	3 Fillis with 0.3 g

